

SPECIMEN SHIPPING INSTRUCTIONS

1. Download and fill out our Specimen Submission form

- ✓ *Fill out all Patient and Clinician information. Also include any pertinent past and current history of the patient. Include number, types, and condition of the samples. Describe what you are looking for, including clinical diagnosis.*

2. Packaging the Specimen

- ✓ *Package your formalin-fixed specimen in a leak-proof container. Seal jars to ensure no liquid can leak out. Wrap containers with bubble wrap or packing peanuts and use chemical spill pads, if available.*

3. Fixation

- ✓ *Specimens should be fixed in 10% neutral buffered formalin at a 10:1 ratio. Do not ship large quantities of formalin, i.e. less than 1 liter is okay.*

4. Quotation & Pricing

- ✓ *Email: info@appliedpathology.com to request a formal quotation. For general inquiries call (508)-796-3875. Wait to receive your quotation before you ship the samples.*

5. Shipping Address

- ✓ *Mailing Address:*
222 Maple Avenue
Rose Gordon Building, Room 222
Shrewsbury, MA 001545
Phone: (508)-796-3875
Fax: 508-644-0418